


Automated External Defibrillator Usage Instructions

1

Turn On the AED

Follow the AED's Voice and Visual Prompts after Turning On

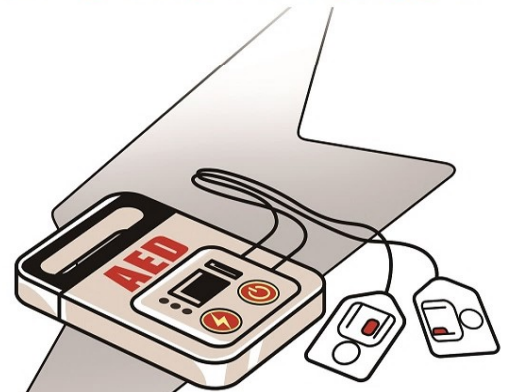


Automated External Defibrillator **AED**
The 'lifesaver' for sudden cardiac arrest patients

2

Attach the Electrode Pads

Place the Electrode Pads on the Chest as Shown

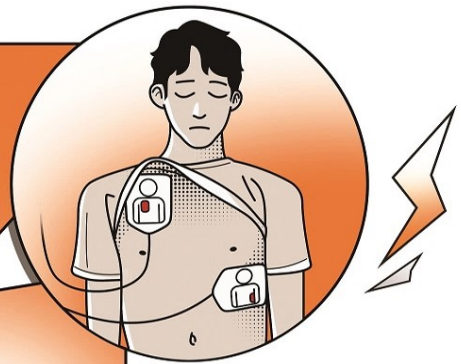

3

Allow AED to Analyze

No Touching of the Patient, Allow AED to Analyze



4

Follow AED's Shock Advisory

If the AED Advises a Shock, Ensure No One is Touching the Patient and Press the Shock Button to Deliver the Shock



Resume CPR as Needed, 2 minutes later, Allow AED to Analyze Again



Red Cross
First Aid By Your Side

Note: Using an AED is simple, and with the right training, even those without a medical background can safely operate it.

Support from the National Lottery Public Welfare Fund
Manufactured by the Chinese Red Cross Training Center